Cerclagen

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Wire Tightener/Twister
Bone Cerclage
Thabe Titanium Cerclage Band

Wire Tightener / Twister

- Description
- Instruments
- Accessories

Bone Cerclage

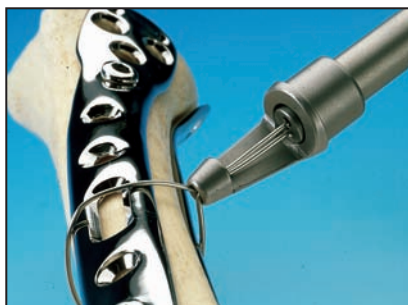
- Description
- Indications
- Application
- Implants
- Instruments

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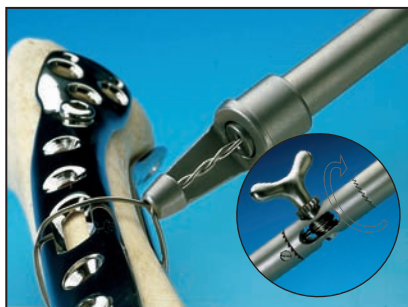
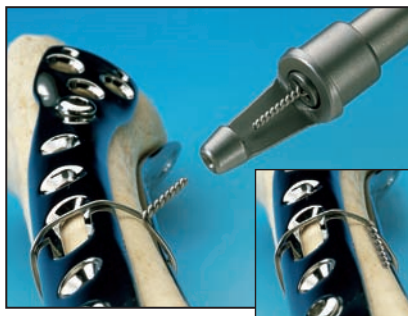
■ LINK® Wire Tightener/Twister

Both ends of the wire are lead around the bone respectively bone plate, entered into the tip of the Wire Tightener and guided through the adjacent tubing of the instrument between the shanks of the wing handle's axis.

For tightening the wire, the wing handle is rotated, thus winding the wire around the shanks of the handle's axis.

For twisting the wire, the distal section of the Wire Tightener is held tight, whilst the proximal section of the instrument is rotated, using the ratchet mechanism. In the distal window just behind the instrument's tip, twisting of the bone wire can be observed.

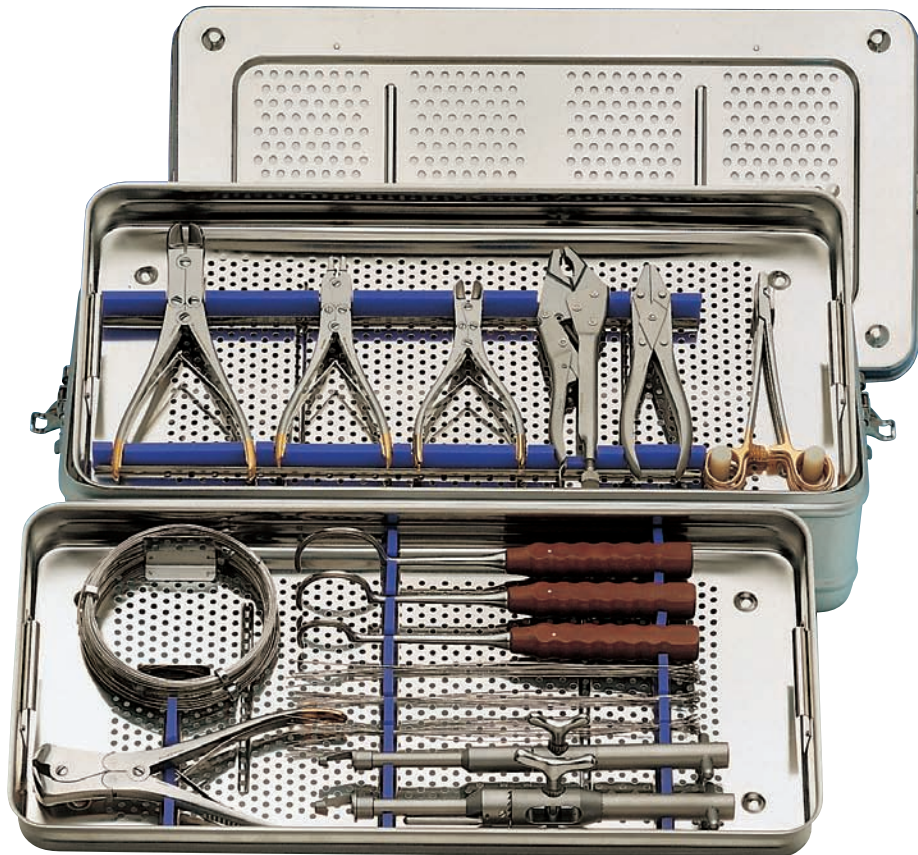
Further (forceful) twisting of the wire results in its desired breakage within the twisted wire section, 15 to 20 mm above the bone in the window just above the tip of the instrument.

After removal of the Wire Tightener/Twister from the operating site, the remaining wire in the instrument can be coiled up by forceful rotation of the instrument's wing handle, thus opening the distally twisted wire. Thereafter the wing handle is removed from its seating, pulling strongly on the handle's wings. When the handle disengages, the coiled wire can be removed from the window under the handle's seating.

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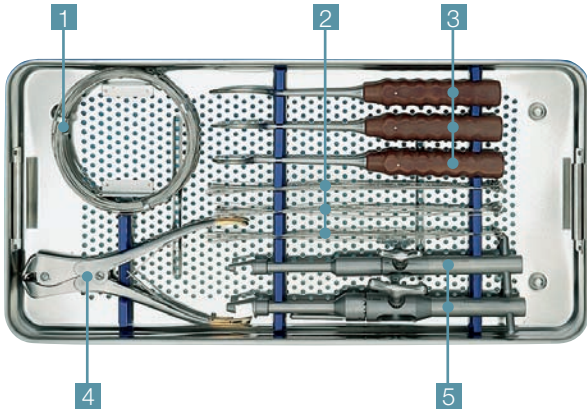
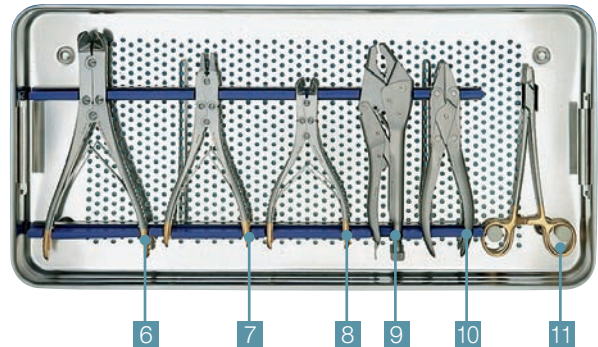
Wire Tightener / Twister
up to Ø 0.8 mm wire, 150 mm

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Cerclage Wire for Wire Tightener / Twister
470 mm, Package of 10

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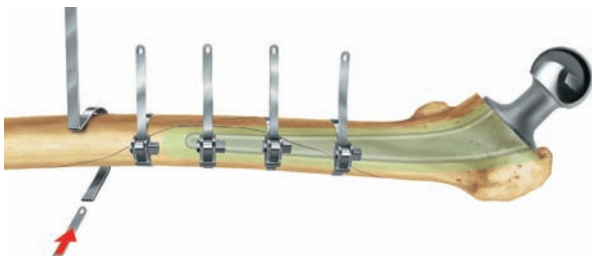
■ LINK® Bone Cerclage

The Cerclage Band with a locking mechanism consists of a band, manufactured from stainless steel, 165 mm long and 5 mm wide and a lock.

The lock is positioned at one end of the band. The other end has a hole for a secure grip and tightening using the Band Tightener Pliers.

The locking mechanism features a toothed gear inside the housing. To prevent any return motion of the tightened Cerclage Band a tooth serves as an abutment. The Cerclage Band is tightened by rotation of a square end axle at the side of the gear using a Spanner Key.

Indications

1. Fractures
2. Fenestrations in revision surgery
3. Infractures
4. Stabilization of thin tubular bone walls
5. Reconstruction using autologous / homologous bone graft
6. Fixation and stabilisation in reconstruction procedures

Surgical Procedure

In order to avoid damage to nerves and vessels the preparation is performed near to the bone using a curved elevator or similar instrument.

The hollow Band Guide which is specially designed for this cerclage technique is guided around the bone. The Cerclage Band is then passed around the bone, inside the Band Guide.

Having the Band Guide removed, the distal end of the Cerclage Band is directed through the toothed gear of the lock and fastened using the Band Tightener Pliers. The final tightening is performed with an angled Spanner Key. The excess end of the Cerclage Band is cut off using the Wire Cutter and then shaped with a Staple Driver to avoid soft tissue irritations from sharp edges.

The lock should always be covered by sufficient soft tissue or placed into a muscle cavity in order to prevent protrusion.

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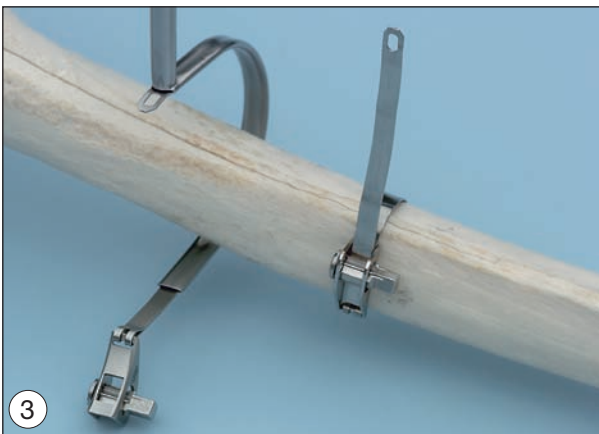
64-4300/10

Cerclage Band with Lock

complete, Length with lock: 165 mm, width 5 mm
Material: Stainless Steel

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Band Tightener Plier
with one tooth and LINK® Handle, 170 mm

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Wire Cutter
angled carbide jaws, double action
220 mm (for wire up to Ø 3 mm)

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Staple Driver
tip hollow and serrated, 5 x 8 mm, 180 mm

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5,8 mm breit, 240 mm, Material: Titan

■ Thabe Titanium Cerclage Band

The Thabe Cerclage Band offers an efficient and easy way to treat fractures of the femur. Compared to wire cerclage systems, the wide contact surface of the band distributes compression forces evenly over a wide area and prevents cutting into the bone. The corrugated portion of the cerclage band preserves the periosteal blood supply, essential to the revitalization of fracture fragments or bone transplants.

Titanium is biocompatible, allowing the cerclage to remain in the body for longer.

The Thabe Titanium Cerclage Band is 5.8 mm wide and 240 mm long (Fig. A). The band is crimped at the lock. The 3.5-cm-long portion facing the bone at the lock is corrugated (Fig. B), thus allowing the periosteal supply to be maintained. One hole at the end of the Cerclage Band receives the pin of the Cerclage Band Pliers and the pin of the Cerclage Band Tighteners.

Fixation of the Thabe Cerclage Band is by turning the headless screw inside the lock (Fig. C), as the tip of the hex screw presses the band into the recess in the bottom of the lock.

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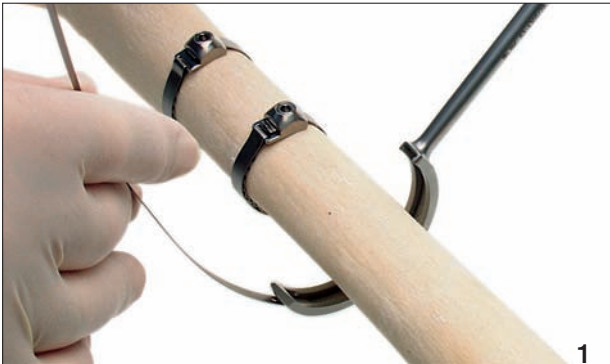
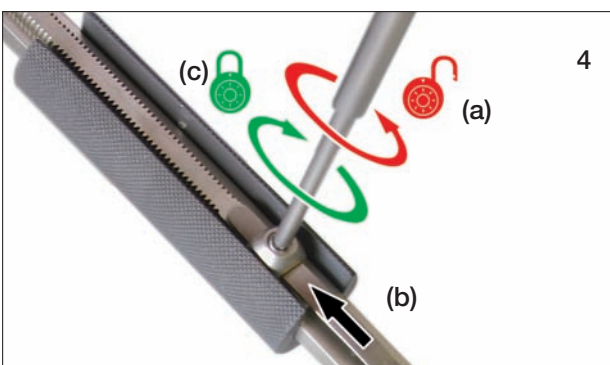
The Cerclage Band Guide of the appropriate size is passed around the bone (Fig.1). From the outside, the end section of the Cerclage Band is slid into the gap at the tip of the instrument and pushed on gently through the inside track until it comes out at the other end. The Guide can now be removed.


The end of the Cerclage Band is then pushed through the anterior slot of the cerclage lock (Fig. 2) and drawn through the slot as far as possible by hand or by using the Cerclage Band Pliers (Fig. 3).

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- a) The slot of the Cerclage Band Tightener has to be opened with the Hex Screwdriver, before inserting the Cerclage Band into the slot.
- b) After insertion, the Cerclage Band is pushed through the slot as far as possible.
- c) The Cerclage Band is secured by tightening the screw in the slot turning the Hex Screwdriver clockwise (Fig.4).

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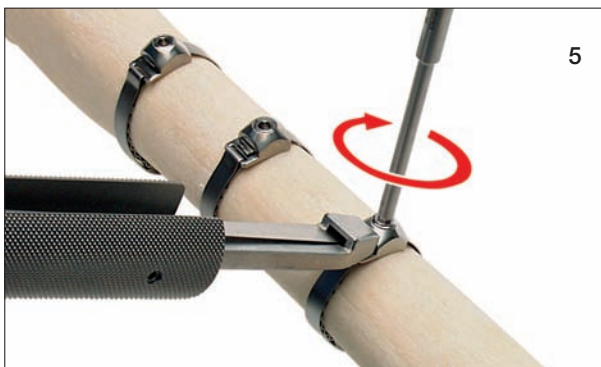Now the Cerclage Band can be tightened by turning the wheel clockwise. The cerclage is secured by tightening the headless screw in the lock using the Hex Screw-driver (Fig. 5).

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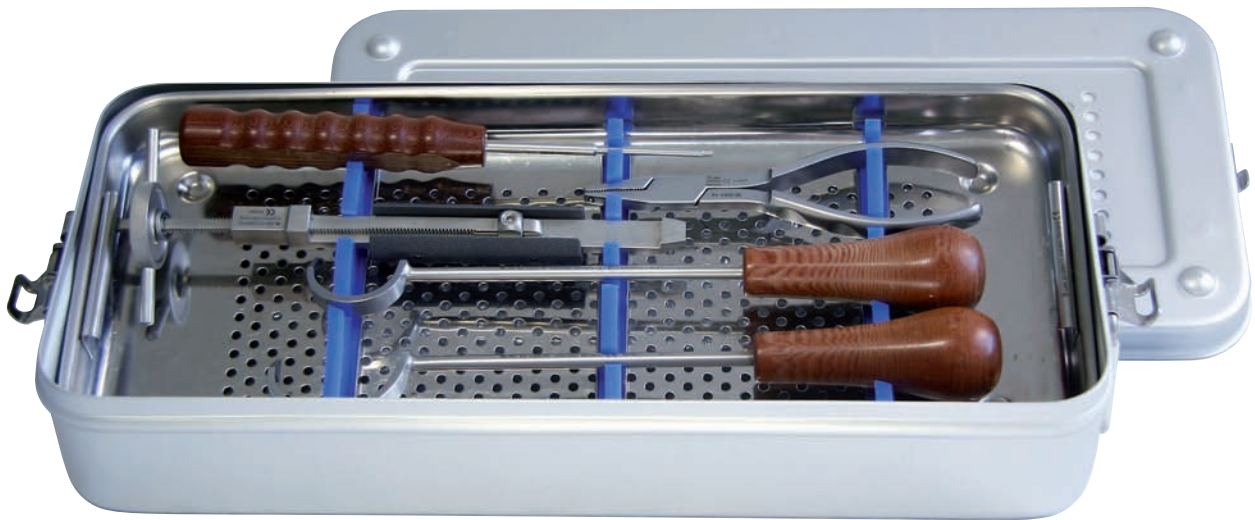
The protruding end of the Cerclage Band is severed close to the lock by moving it rapidly back and forth (Fig. 6a, 6b), so as to avoid soft tissue irritation from contact with a projecting sharp metal edge (Fig. 7).

63-4300/05	Instrumentarium für Cerclage n. Thabe, komplett	Instrument Set for Thabe Cerclage, complete
	bestehend aus:	consisting of:
05-1000/01	Kleincontainer K1 inkl. Pack á 5 Papierfilter, 460 x 190 x 92 mm	K1 Small Container incl. Pack of 5 paper filters, 460 x 190 x 92 mm
63-4300/11	Siebeinsatz , leer 415 x 175 x 35 mm	Tray , empty 415 x 175 x 35 mm
63-4300/19	Cerclageband-Umführungsinstrument Innenradius 40 mm, 290 mm	Cerclage Band Guide inner radius 40 mm, 290 mm
63-4300/20	Cerclageband-Umführungsinstrument Innenradius 50 mm, 300 mm	Cerclage Band Guide inner radius 50 mm, 300 mm
63-4300/40	Cerclageband-Spanngerät	Cerclage Band Tightener
175-600	Sechskantschraubendreher SW 3,0 mm, 230 mm	Hex Screwdriver hex 3,0 mm, 230 mm
64-4300/30	Cerclage-Spannzange mit Zahn und LINK® Handformgriff, 170 mm	Band Tightener Plier with one tooth and LINK® Handle, 170 mm

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Bone Cerclage

Item no.	Page
B	
Band Guides	8
Band Tightener Pliers	8
C	
Cerclage Band with Lock	7
Container K1, Small	8
I	
Instrument Set for Bone Cerclage	8
S	
Spanner Key	8
Staple Driver	8
Sterilizing Tray, I lower and II upper	8
W	
Wire Cutter	8

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Please note the following regarding the use of our implants:

- 1. Choosing the right implant is extremely important.**
The size and shape of the human bone determine the size and shape of the implant and also limit the load capacity. Implants are not designed to withstand unlimited physical stress. Demands should not exceed normal functional loads.
- 2. Correct handling of the implant is exceedingly important.**
Under no circumstances should the shape of a finished implant be altered, as this shortens its life span. Our implants must not be combined with implants from other manufacturers.

The instruments indicated in the Surgical Technique must be used to ensure safe implantation of the components.
- 3. Implants must not be reused.**
Implants are supplied sterile and are intended for single use only. Used implants must not be reused.
- 4. After-treatment is also very important.**
The patient must be informed of the limitations of the implant. The load capacity of an implant cannot compare with that of healthy bone!
- 5. Unless otherwise indicated, implants are supplied in sterile packaging.**
Note the following conditions for storage of packaged implants:
 - Avoid extreme or sudden changes in temperature.
 - Sterile implants in their original, intact protective packaging may be stored in permanent buildings up until the "Use by" date indicated on the packaging.
 - They must not be exposed to frost, dampness and direct sunlight or mechanical damage.
 - Implants may be stored in their original packaging for up to 5 years after the date of manufacture. The "Use by" date is indicated on the product label.
 - Do not use an implant if the packaging is damaged.
- 6. Traceability is important.**
Please use the documentation stickers provided to ensure traceability.
- 7. Further information** on the material composition is available on request from the manufacturer.

Follow the instructions for use!

WALDEMAR LINK GmbH & Co. KG, Hamburg.

All content in this catalogue, including text, pictures and data, is copyright-protected. Every instance of use not permitted by the German Copyright Act is subject to our prior consent. In particular, this applies to the reproduction, editing, translation, saving, processing or passing on of content stored in databases or other electronic media and systems. The information in this catalogue is solely intended to describe the products and does not constitute a guarantee.

The Surgical Technique described has been written to the best of our knowledge and belief, but it does not relieve the surgeon of his responsibility to duly consider the particularities of each individual case.

Unless otherwise indicated, all instruments are made of surgical stainless steel.

WALDEMAR LINK GmbH & Co. KG
Barkhausenweg 10 · 22339 Hamburg, Germany
P.O. Box 63 05 52 · 22315 Hamburg, Germany
Tel.: +49 40 53995-0 · Fax: +49 40 5386929
E-mail: info@linkhh.de · Internet: www.linkhh.de